


Tilting Toilet lift

INSTALLATION MANUAL

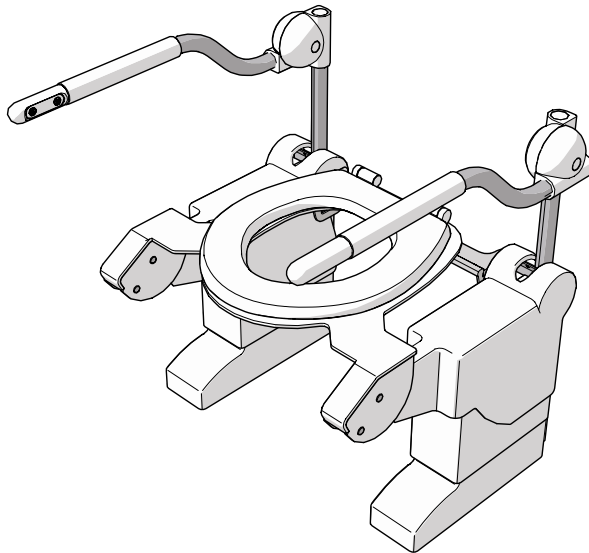

ABOUT THIS MANUAL

This manual contains instructions and safety information for installation and maintenance of the Aerolet® tilting toilet lift

Language

The original manual is written in UK English. All other language versions are translations of the original manual.



Intended readers

This manual is intended for:

- The technician that installs or performs maintenance on the machine

Symbols used

This manual contains safety instructions that may result in injury when ignored. Each safety warning is indicated with a signal word. The signal word corresponds with the level of risk of the described hazardous situation:

Safety symbol	Level of risk	Result if not avoided
 WARNING	Medium	Could result in death or serious injury.
 CAUTION	Low	Could result in moderate or minor injury or damage to the machine.

Safety warnings given at the start of a section apply to the entire section.

Other symbols used

This manual contains symbols that indicate useful information that complements the instructional text and illustrations.

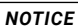
Symbol	Meaning
 NOTICE	Messages that are not hazard-related are indicated with the signal word NOTICE. These messages do not have a safety alert symbol.

TABLE OF CONTENTS

1. Product	4
1.1. Intended use	4
1.2. Reasonably foreseeable unintended use	4
1.3. General specifications	4
2. Description	6
2.1. Main parts	6
2.2. Controls	9
3. Safety	12
3.1. Personal protective equipment	12
3.2. Safety features	12
3.3. Safety symbols and labels on the product	12
3.4. Safety warnings and regulations	12
4. Transport and storage	14
5. Installation	14
5.1. Remove packaging	14
5.2. Contents of the package	14
5.3. Requirements for the location of installation	17
5.4. Preparation	17
5.5. Mounting	20
5.6. Add-ons	34
5.7. Adjusting	34
6. Use	40
6.1. Lifting and lowering with drive unit controls	40
6.2. Resetting	40
7. Troubleshooting	40
8. Maintenance	41
8.1. Repair	41
9. Decommissioning	41
10. Warranty	42
11. Disposal	42
Attachments	43

1. PRODUCT

1.1. Intended use

The essential performance of the toilet lift is to:

- Assist individuals with reduced mobility in using the toilet. The device helps to lower and lift the user on the toilet.
- Assist caregivers who help individuals with reduced mobility to make use of the toilet.

The essential performance of the toilet lift must be what it is intended to be and what the end user can expect. If the intended purpose of the essential performance is lost or deteriorates, the product is no longer suitable for the user.


1.2. Reasonably foreseeable unintended use

The following is considered foreseeable misuse:

- Use of the machine that deviates from the intended use as described in the previous section.
- Failure to comply with the instructions in this manual.
- Failure to eliminate faults, malfunctions or defects of the machine that impose safety risks.
- Failure to carry out the inspections and maintenance operations as described in this manual.
- Unauthorised removal or modification of parts or safety components of the machine.
- Use of spare parts or accessories that have not been approved by the manufacturer.

1.3. General specifications

The most important specifications can be found on the type plate. The type plate can be found on the right hand side, at the back of the toilet frame.

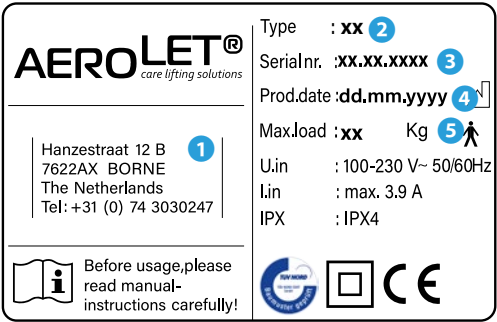

Example of a type plate

- | | |
|--------------------------------|------------------------|
| 1 Contact details manufacturer | 4 Production date |
| 2 Type code | 5 Maximum nominal load |
| 3 Serial number | |


The table below is a full overview of all the technical specifications:

Specification	Value
Name of the machine	Aerolet® tilting toilet lift
Length, width and height	68 x 70 x 89 cm
Weight	30 kg
Voltage	100-230 V
Frequency	50/60 Hz
Capacity	Max. 3.9 A
Working speed	55 mm/sec
Sound level	3.5 dB
Operating and storage temperature	5 – 40 °C
Operating and storage humidity	20 - 60%
Operation altitude	Max. 2000 m
Warranty	2 years

2. DESCRIPTION

2.1. Main parts

This is an overview of the main parts of the Aerolet® tilting toilet lift:

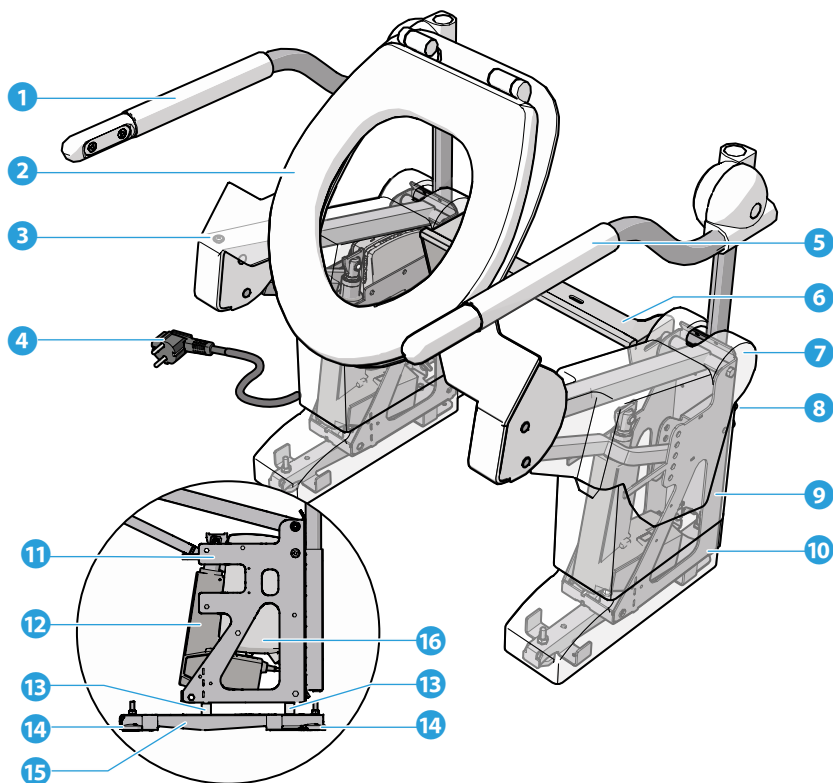


- 1 Right arm support
- 2 Toilet seat
- 3 Top cover locking screw
- 4 Power plug
- 5 Left arm support
- 6 Connecting bracket
- 7 Top cover
- 8 Allen bolts

- 9 Middle cover
- 10 Foot cover
- 11 Drive unit
- 12 Actuator
- 13 Height adapter
- 14 Levelling feet
- 15 Floor frame
- 16 Control box

The diagonal toilet lift is equipped with one of four types of armrests:


Fixed armrests

- This type of armrest is not foldable and can be adjusted in height.
-

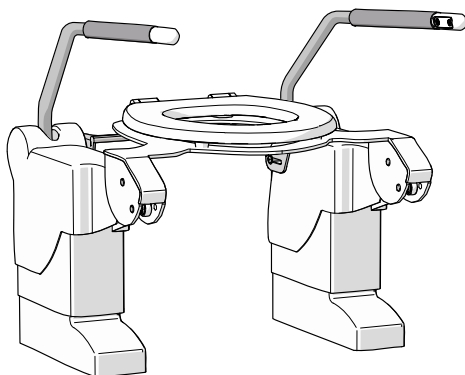


Folding armrests

- This type of armrest can fold upwards and can be adjusted in height.
-

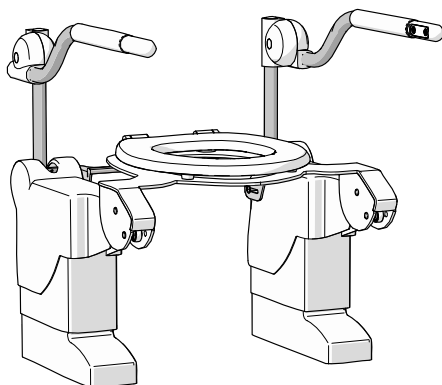


Movable armrests

- This type of armrest tilts alongside the toilet seat and has a fixed height.

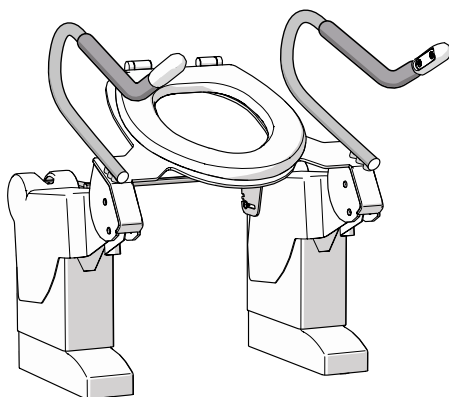

Movable-folding armrests

- This type of armrest tilts alongside the toilet seat, can fold upwards and has a fixed height.

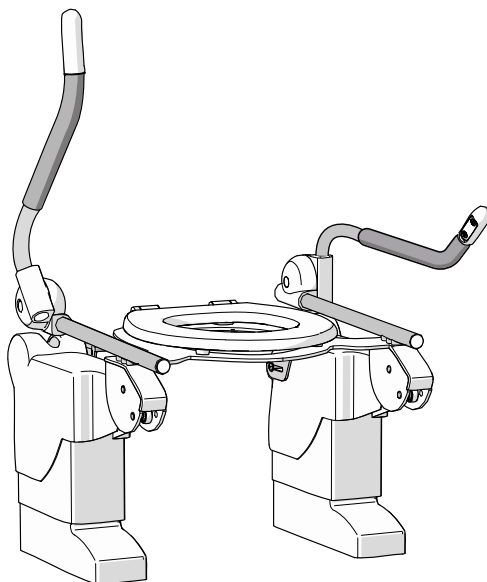

The following add-ons are available:


Battery back-up

- The toilet lift can be equipped with a battery backup. This battery backup ensures that the lift can operate in the event of a power failure.

Bidet system


- If the type code (see type plate) is BI-ST, BI-SM or BI-BA, the toilet lift can be equipped with an on-top mounted bidet system. The bidet-system can be bought and installed by an external company. To prepare the toilet lift for an on-top mounted bidet system, see chapter 5.4.3..

2.2. Controls

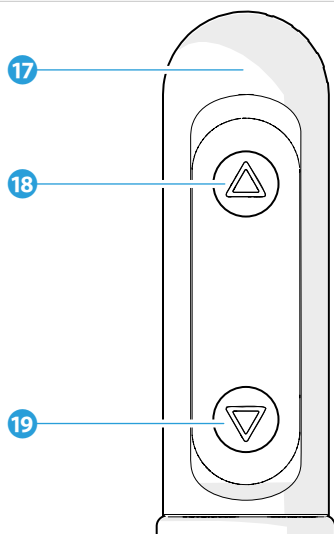

- 17 Arm support control
- 18 Arm support up-button

- 19 Arm support down-button

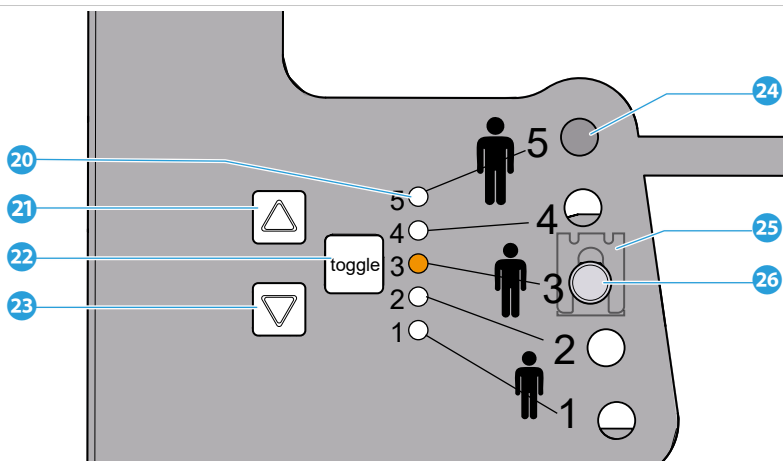


- 20 LED
- 21 Drive unit up-button
- 22 Toggle button
- 23 Drive unit down-button

- 24 Positioning hole
- 25 Locking spring
- 26 Hinge pin

The following control add-ons are available:

Easy-to-use button

- The regular armrest control buttons can be replaced for push buttons at the front of the armrests. This is useful for users with reduced muscle strength or hand function.

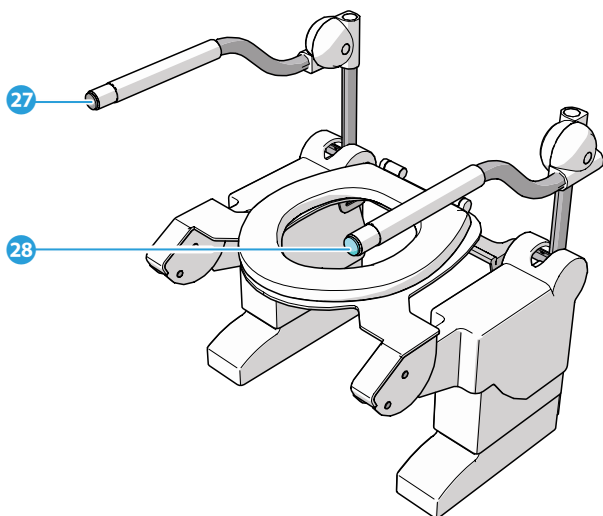


- 27 Easy-to-use down-button

- 28 Easy-to-use up-button

Hand remote control

- Aside from the regular armrest control buttons, a hand remote can be installed. This is useful for caregivers to operate the toilet lift.

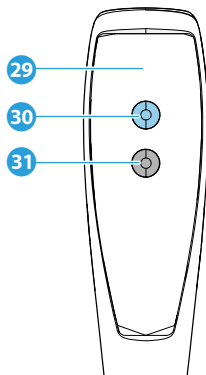


- 29 Hand remote

30 Hand remote up-button
- 31 Hand remote down-button

Foot control

- Aside from the regular armrest control buttons, a foot control can be installed. This is useful for users with reduced hand and/or finger mobility.

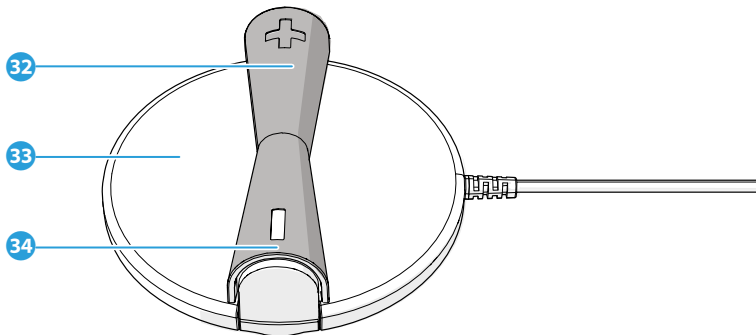

- 32 Foot control up-button

33 Foot control
- 34 Foot control down-button

3. SAFETY

3.1. Personal protective equipment

During installation and maintenance, it is recommended to wear rubber gloves to protect against harmful bacteria and chemicals.

3.2. Safety features

To minimize the risk of a fire hazard or electric shock, the device is equipped with short-circuit protection, thermal protection, hot-plug protection and overload protection.

To protect the device against water damage, and to protect the user from electric shock, the device is equipped with a plastic seating cover and foot cover.

To warn the user when there is an electronics error or if the battery back-up (add on) is empty, the device makes a constant beep sound.

3.3. Safety symbols and labels on the product

This is an overview and explanation of all inflicted safety symbols and labels on the product.



Warning: Fully lower the toilet lift after use.



Warning: Risk of entrapment



Maximum weight indication of 150 kg / 330 lbs.



Maximum weight indication of 250 kg / 550 lbs.

3.4. Safety warnings and regulations

3.4.1. General safety warnings

⚠ WARNING

- Risk of serious injury. Ensure you have fully read and understood the instructions in this document before you install or use the product. Keep the packaging and this document for future reference.
- Risk of serious injury. Always use the product as described in this document.
- Risk of serious injury. Never use the product if a part is damaged or defective. Replace a damaged or defective product or part immediately. Always check the product for damage before use.

⚠ CAUTION

- Risk of product damage. High temperatures can cause overheating. Never operate the lift in temperatures above 40 degrees Celsius.
- Risk of product damage. Excessive humidity can cause electrical issues and product malfunction. Never operate the lift when humidity exceeds 60%.
- Risk of product damage. Always use the specified power supply. Ensure you use the correct power specifications.

3.4.2. Safety warnings for transport

⚠ WARNING

- Risk of product damage. Never drop or handle the lift roughly during transport. Handle with care.

3.4.3. Safety warnings for installation

- Improper installation hazard. Risk of serious injury and product malfunction. Always install the device by trained and authorized personnel. Follow the installation instructions carefully.
- Electrical hazard. Risk of fire or serious injury. Risk of short-circuiting or unstable product due to improper installation. Follow all local electrical safety codes when installing the lift. Adhere to safety regulations.
- Improper hardware hazard. Risk of product malfunction and serious injury. Use only the mounting hardware mentioned in this manual.
- Product instability hazard. Risk of serious injury due to a loose toilet bowl. A loose toilet bowl can result in an unstable toilet lift. Always anchor the toilet bowl solidly to the floor or wall. Always ensure the lift is installed on a stable and level surface. Verify the surface before installation.
- Hand entrapment hazard. The seat frame will tilt backwards when the hinge pins are removed. Handle carefully.
- Product instability hazard. Risk of serious injury due to a loose or cracked toilet seat. Ensure the seat is securely fixed. Only use the recommended toilet seat with the correct fixing materials.

⚠ CAUTION

- Risk of product malfunction. High altitudes can affect the lift's performance and safety. Never install the device above an altitude of 3000 meters.
- Risk of product malfunction. Testing ensures the lift is functioning correctly and safely. Always perform a lift operation test after installation.
- Risk of product damage. Ensure there is enough space between the wall and the toilet seat to allow for the tilting motion. The toilet seat must never touch or bump into the wall.
- Risk of product damage or improper operation. Never install a toilet lid on the toilet lift. The lid may obstruct the tilting motion and cause mechanical interference.
- Risk of product damage. Use the same hinge hole positions on both sides. Uneven hinge pin positions may result in serious mechanical damage to the toilet lift.
- Risk of damage to the armrests. Never adjust the armrests to a position where the toilet seat touches the armrests. Always leave enough room for the armrests to move up and down to prevent surface scratches.

3.4.4. Safety warnings for maintenance

⚠ WARNING

- Risk of electrical shock. Disconnect the power supply before performing any maintenance. Always unplug the device first.
- Improper maintenance hazard. Risk of serious injury and product malfunction. Always repair the device by trained and authorized personnel. If the toilet lift is not repaired by an authorised body, there is a risk of malfunction and the warranty may be withdrawn.

4. TRANSPORT AND STORAGE

CAUTION

Risk of product damage. Never drop or handle the lift roughly during transport. Handle with care.

The toilet lift is delivered by Aerolet on a pallet in a protected package which weighs 30 kg. It is recommended to:

- Transport the toilet lift in an upright position
- Transport in the provided packaging box to protect the product from dust and debris.

5. INSTALLATION

WARNING

Improper installation hazard. Risk of serious injury and product malfunction. Always install the device by trained and authorized personnel. Follow the installation instructions carefully.

5.1. Remove packaging

To remove the packaging material from the toilet lift, gently cut along the taped seams with a utility knife, taking care not to press too hard to avoid scratching the surface. Slowly peel away any protective plastic foil or foam cushioning.


5.2. Contents of the package

Check if all parts are present and undamaged. In case of missing or damaged parts, please contact your supplier.

1. Tilting toilet lift
2. Height adapter bushes, (if applicable)
3. Mounting set 1, 2, 3, or 4
4. Wrench size 13
5. Installation checklist


Depending on the type of toilet bowl, one of the following mounting sets is supplied:

Mounting set 1 – Suitable for conventional toilet bowl

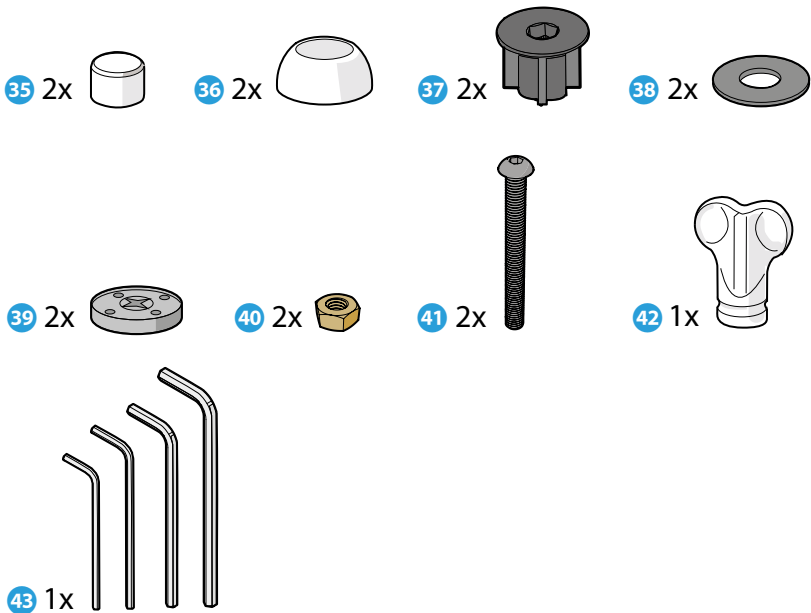


- | | |
|---------------------|---------------------|
| 35 Bolt cover cap | 40 M6 nut |
| 36 Bottom cover cap | 41 M6 80 mm bolt |
| 37 Centering bush | 42 Mounting key |
| 38 Rubber ring | 43 Allen wrench set |
| 39 Lock washer | |

Mounting set 2 – Suitable for Geberit Mera toilet bowl

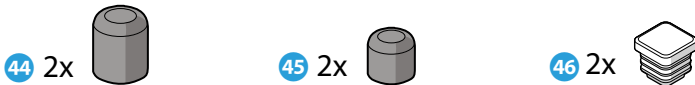


- | | |
|------------------------|---------------------------|
| 44 Large round end cap | 46 Toilet lid sealing cap |
| 45 Small round end cap | |

Mounting set 3 – Suitable for Closomat Palma toilet bowl

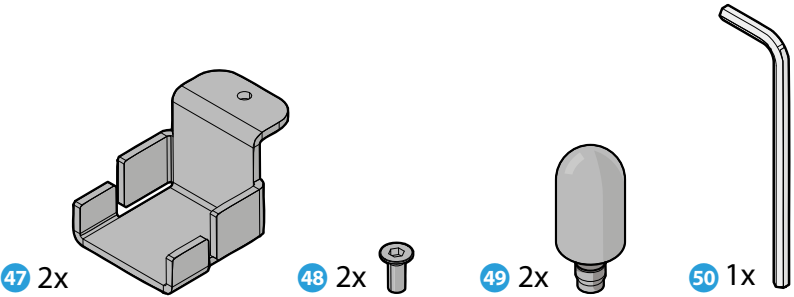


- 47 Anchor plate
- 48 Anchor plate bolt
- 49 Seat lugs
- 50 Allen wrench
- 51 10 mm spacer

Mounting set 4 – Suitable for JHC/Washair toilet bowl

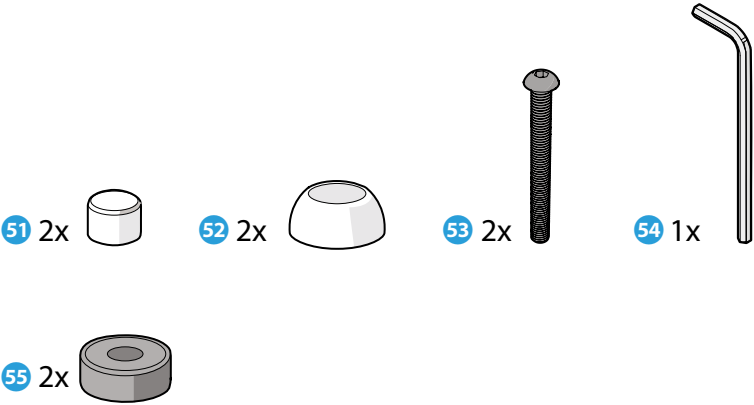

- 52 Bolt cover cap
- 53 Bottom cover cap
- 54 M6 60 mm bolt
- 55 Allen wrench

5.3. Requirements for the location of installation

 **CAUTION**

Risk of product malfunction. High altitudes can affect the lift's performance and safety.
Never install the device above an altitude of 3000 meters.

The location of installation has the following requirements:

- One 230V Power outlet within 2 meters of the toilet bowl. If the power outlet is not within range, use an extension cord.
- A toilet bowl, securely connected to the floor or wall and level with the floor.
- Additional power outlet and water point within 2 meters in case of the bidet system add-on.


5.4. Preparation

5.4.1. Preparation of the installation site


Check if the location of installation has a 230 V Power outlet within 2 meters of the toilet bowl. If not, plug in an extension cord.

Check that the installation site has the following minimum dimensions for placement of the toilet lift:

Measurement	Measurement requirement
Distance A	Min. 105 mm
Distance B	Min. 50 mm
Distance C	Min. 100 mm

 **CAUTION**

Risk of product damage. Ensure there is enough space between the wall and the toilet seat to allow for the tilting motion. The toilet seat must never touch or bump into the wall.

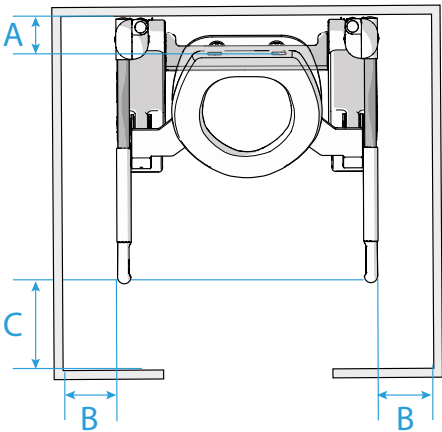

5.4.2. Preparation of toilet bowl

Remove the toilet seat from the toilet bowl. Consult the manual of your specific toilet.

5.4.3. Preparation of toilet lift

Measure the height of the toilet bowl. Is the height more than 400 mm, install a toilet raising set.


Install toilet raising set

NOTICE




Scan the QR code to access all Aerolet's installation videos online.

1. Carefully tilt the toilet lift back and ensure that the floor frames **15** are clear of the ground. Use a pair of support objects, such as wooden blocks, to support the toilet lift.
2. Open the velcro connection on the left and right foot covers **10** and push the foot covers slightly upwards.
3. Remove the floor frame bolts and the floor frame **15**.

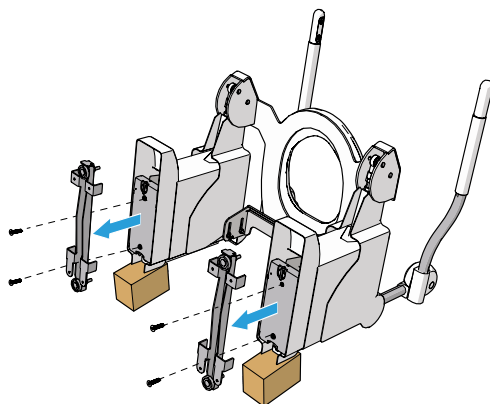

4. Attach the height adapter bushings to the floor frame 15 with bolts as depicted in the image.
-

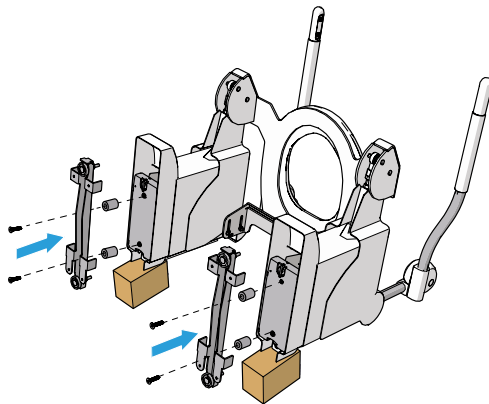


5. Slide foot cover 10 back over floor frame 15 and press the velcro connections closed.
6. Carefully set the toilet lift upright and continue with the installation procedure.
-

Install nylon stoppers for on-top mounted bidet

To prepare the toilet lift for an on-top mounted bidet system, two nylon stoppers should be installed. The stoppers prevent the bidet from sliding forward during the tilting motion of the toilet seat.

To install the stoppers:

1. Push the two bolts through the bottom of the toilet seat.
 2. Place the bushings on the bolts and place the cap nuts on the bushings. Screw the cap nuts tight.
-

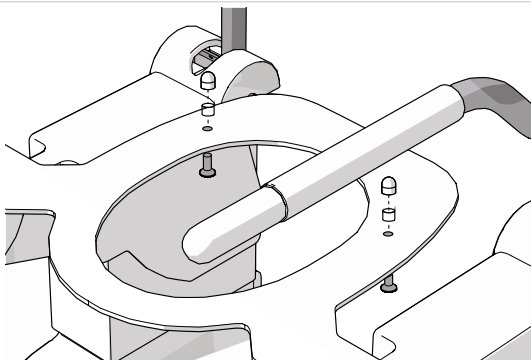

5.5. Mounting

⚠ WARNING

Product instability hazard. Risk of serious injury due to a loose toilet bowl. A loose toilet bowl can result in an unstable toilet lift. Always anchor the toilet bowl solidly to the floor or wall. Always ensure the lift is installed on a stable and level surface. Verify the surface before installation.

⚠ CAUTION

Risk of product damage or improper operation. Never install a toilet lid on the toilet lift. The lid may obstruct the tilting motion and cause mechanical interference.


5.5.1. Mounting toilet lift on a conventional toilet bowl

NOTICE




Scan the QR code to access all Aerolet's installation videos online.


1. Push the foot covers  slightly upwards.

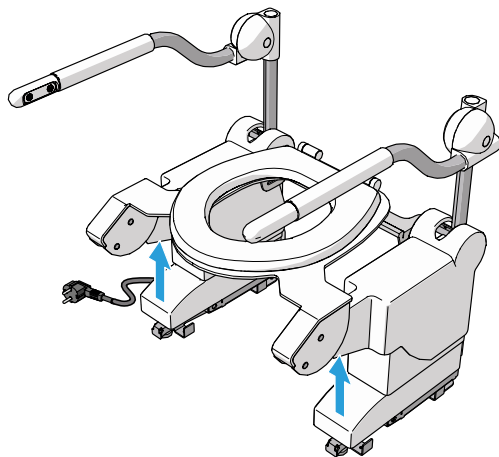


2. Connect the power plug 4 to a power outlet.
 3. Raise the toilet lift approximately 10 cm by pushing the arm support up-button 18.
-

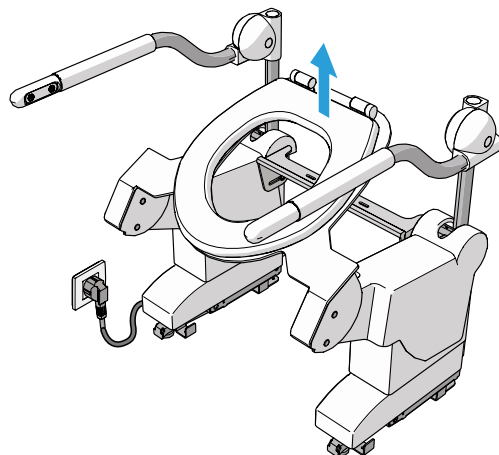


4. Press two centering bushes 37 into the toilet bowl fixing holes. Cut the centering bushes to size if necessary.
-

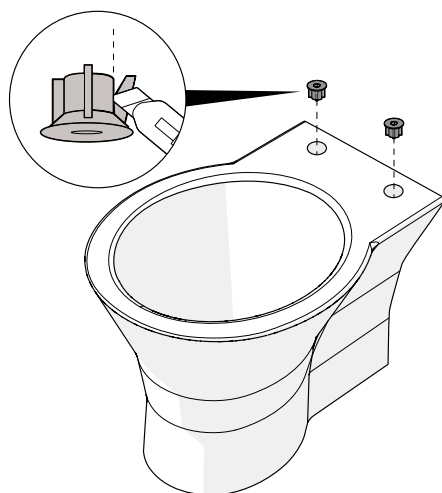

5. Lift the toilet lift by the arm supports. Slide the toilet lift over the toilet bowl and align the slotted holes in the connecting bracket 6 with the centering bushes.
 6. Adjust the levelling feet 14 until the toilet lift is level with the toilet.
 7. Adjust the depth of the lift by aligning the front of the toilet lift with the front of the toilet bowl and tighten the connecting bracket bolts.
-

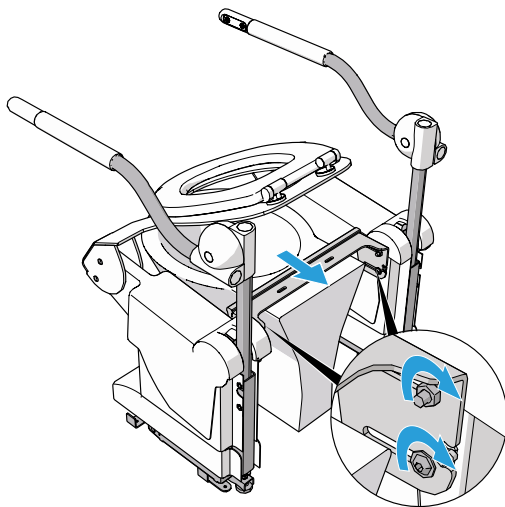

8. Place the rubber rings **38** between the connecting bracket and the toilet bowl.
 9. Push two M6 80mm bolts **41** with bottom cover caps **36** through the connecting bracket, rubber dampers and centering bushes.
 10. Place lock washers **39** on the underside of the toilet bowl and fasten the M6 80mm bolts to the connecting bracket with an M6 nut **40**.
 11. Cover the M6 80mm bolt with a bolt cover cap **35**.
-

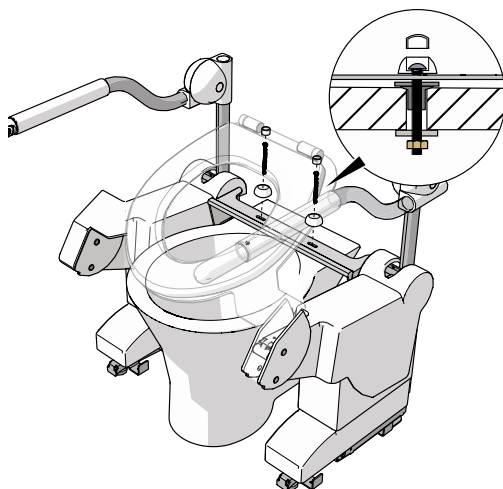

-
12. Lower the foot covers and fasten the velcro connections.

5.5.2. Mounting toilet lift on a Geberit Mera


NOTICE




Scan the QR code to access all Aerolet's installation videos online.

1. Remove the toilet seat and lid from the Geberit Mera. Refer to the Geberit Mera instruction manual for detailed instructions.


2. Push toilet lid sealing caps 46, small round end caps 45 and large round end caps 44 in the holes left by the toilet lid as indicated in the illustration.
-

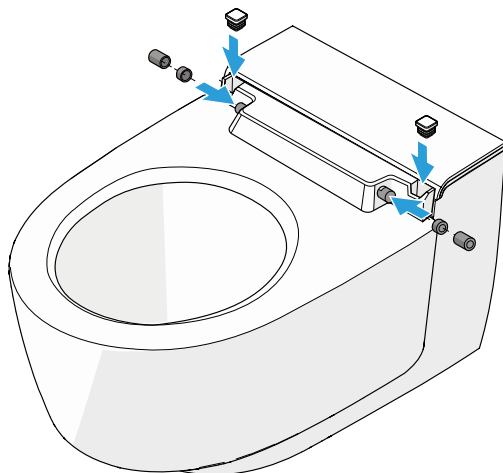


3. Unscrew the plastic set screws on the connecting bracket 6.
-

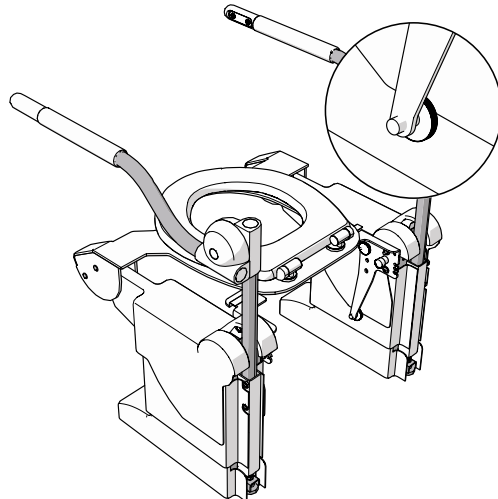


4. Push the foot covers **10** slightly upwards.
-

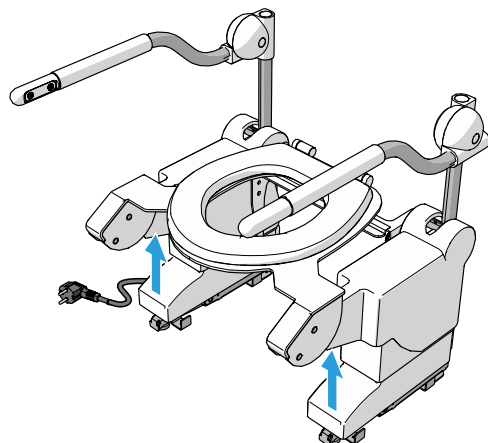


5. Connect the power plug **4** to a power outlet.
6. Raise the toilet lift approximately 10 cm by pushing the arm support up-button **18**.
-

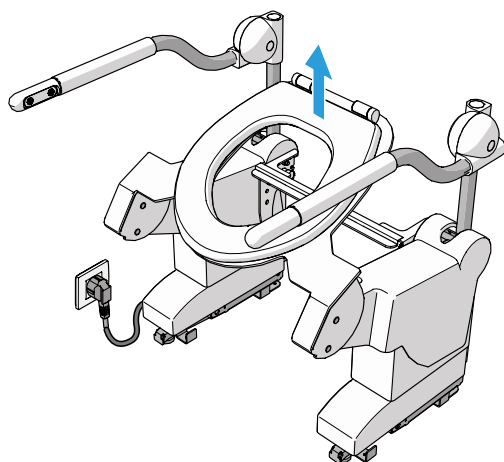

7. Lift the toilet lift by the arm supports ① ⑤. Place the toilet lift over the Geberit Mera and slide through until the cams of the connecting bracket fall behind the toilet bowl.
-

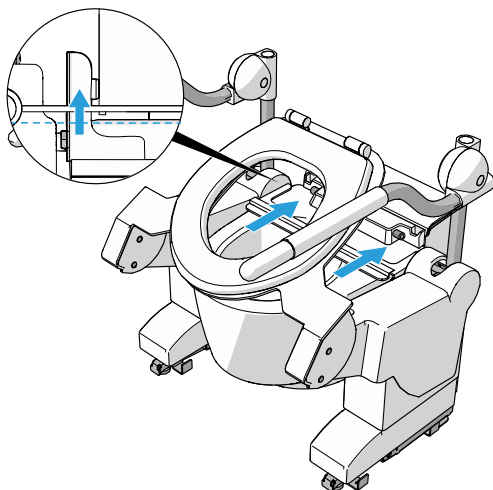

8. Adjust the levelling feet ⑭ until the toilet lift is level with the toilet.
9. Tighten the plastic set screws against the toilet bowl on either side.
-

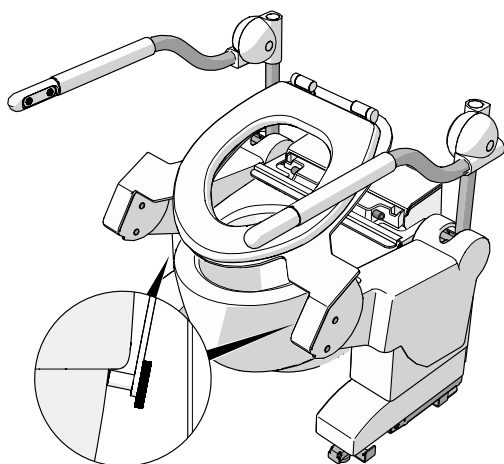


10. Lower the foot covers and fasten the velcro connections.
11. If desired, place the toilet seat back on the toilet lift.


5.5.3. Mounting toilet lift on a Closomat Palma

NOTICE




Scan the QR code to access all Aerolet's installation videos online.


1. Remove the toilet seat and lid from the Closomat Palma. Refer to the Closomat Palma instruction manual for detailed instructions.
 2. Remove the top cover from the Closomat Palma. Refer to the Closomat Palma instruction manual for detailed instructions.
 3. Place the seat lugs  on the holes left by the toilet seat.
-

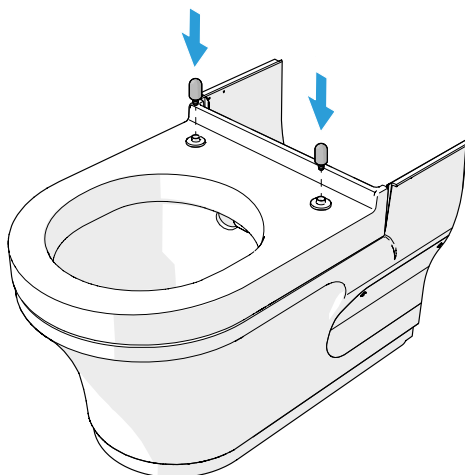


4. Hook the anchor plates 47 behind the toilet bowl.
-

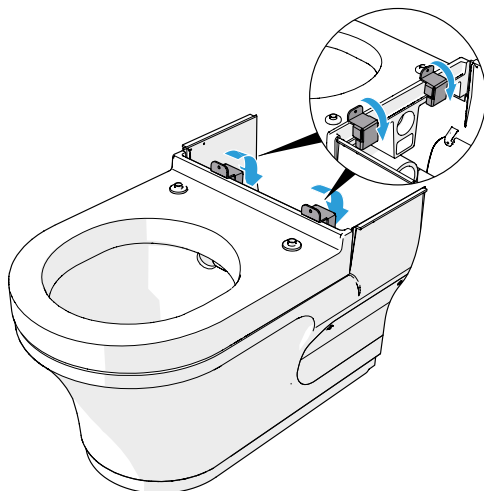


5. Push the foot covers 10 slightly upwards.
-

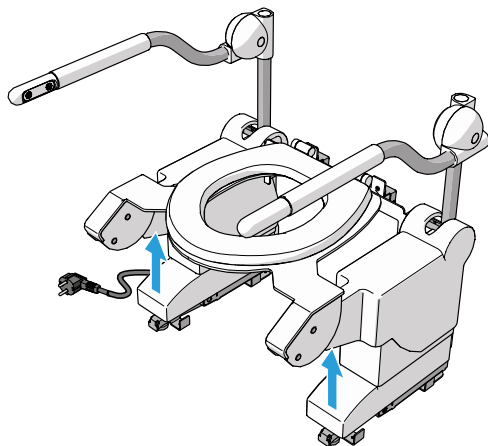


6. Connect the power plug ④ to a power outlet.
 7. Raise the toilet lift approximately 10 cm by pushing the arm support up-button ⑮.
-

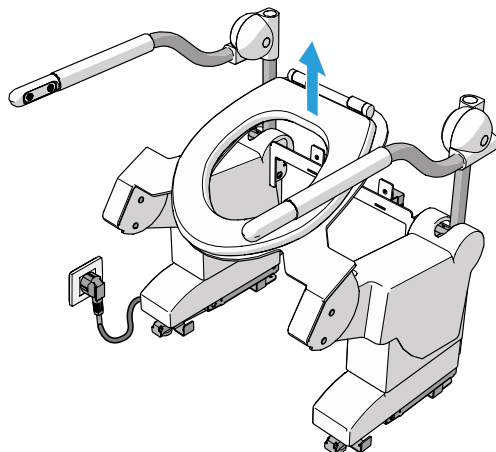

8. Lift the toilet lift by the arm supports ① ⑤. Slide the toilet lift over the toilet bowl and align the holes of the connecting bracket ⑥ with the holes on the anchor plates.
 9. Adjust the levelling feet ⑭ until the toilet lift is level with the toilet.
 10. Attach the anchor plates onto the connecting bracket with the anchor plate bolts ④⑧.
-

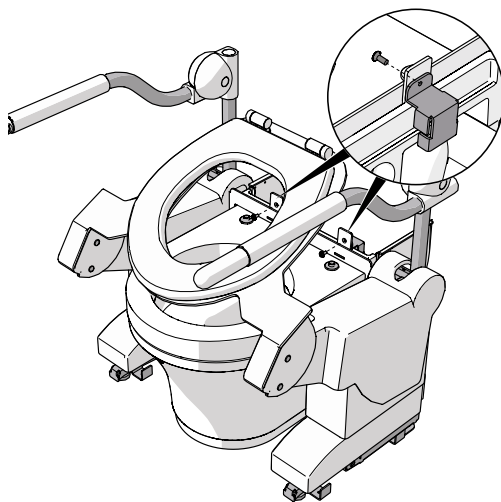

11. Re-install the top cover. Refer to the Closomat Palma instruction manual for detailed instructions.
12. Lower the foot covers and fasten the velcro connections.

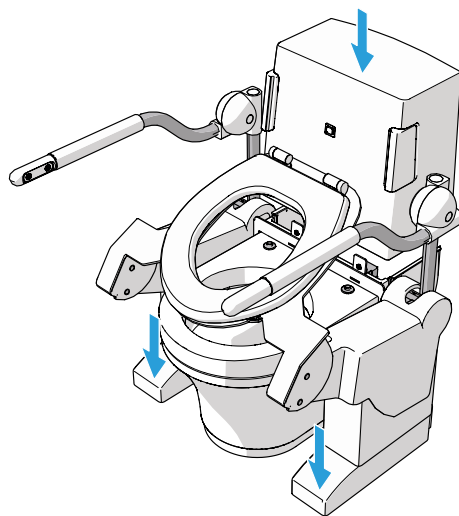

5.5.4. Mounting toilet lift on a JHC/Washair

NOTICE




Scan the QR code to access all Aerolet's installation videos online.


Avoid activation of the bidet function. Never use the remote control while installing the toilet lift.

1. Remove the toilet seat and lid from the JHC or Washair toilet. Refer to the JHC or Washair instruction manual for detailed instructions.


2. Remove the toilet seat axes from the toilet bowl with pliers.
-

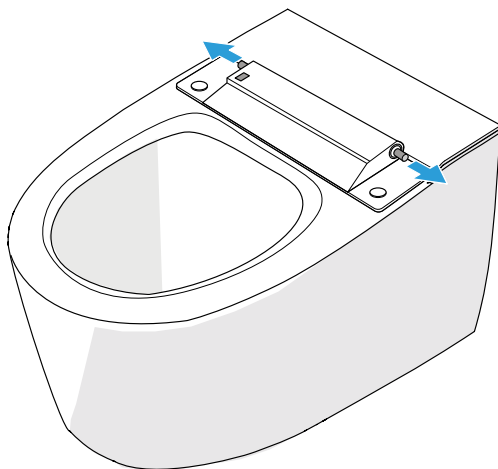


3. Remove the cover caps located on top of the toilet and push the cover caps in the holes left by the toilet seat axes.
-

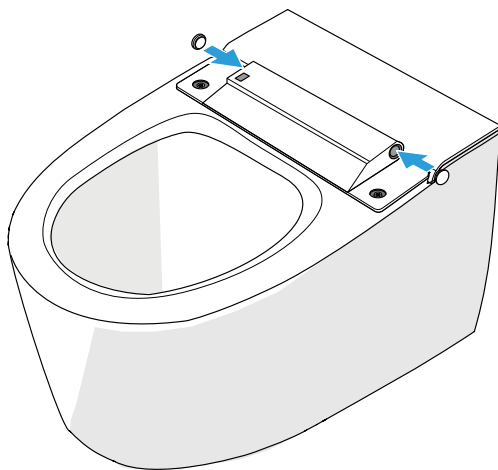


4. Remove the bidet fastening screws.

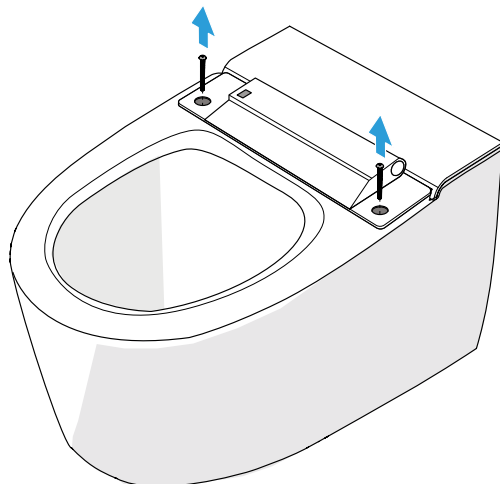


5. Push the foot covers 10 slightly upwards.

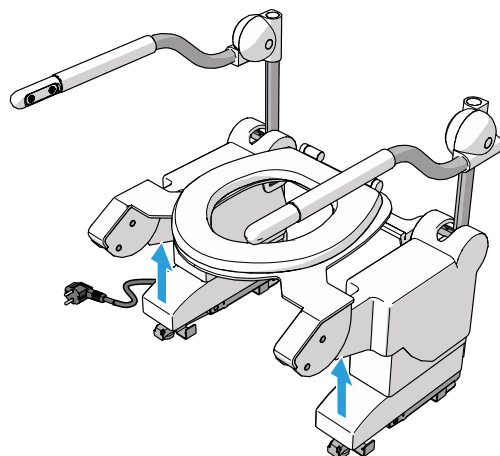

6. Connect the power plug 4 to a power outlet.
 7. Raise the toilet lift approximately 10 cm by pushing the arm support up-button 18.
-

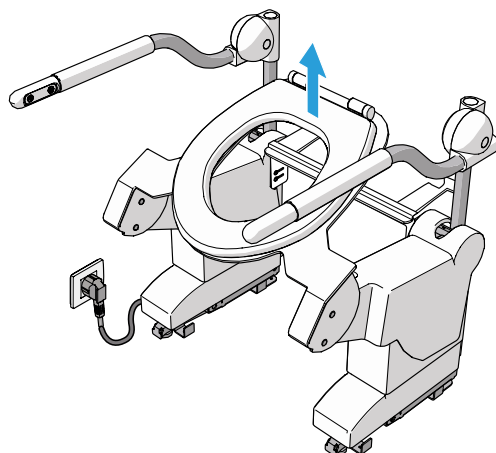

8. Lift the toilet lift by the arm supports 1 5. Slide the toilet lift over the toilet bowl and align the holes of the connecting bracket 6 with the holes on the bidet.
 9. Adjust the levelling feet 14 until the toilet lift is level with the toilet.
 10. Screw the M6 60mm bolt 54 with the bottom cover cap 53 and 10 mm spacer 51 in the bidet holes.
 11. Push the bolt cover cap 52 in the bottom cover cap.
-

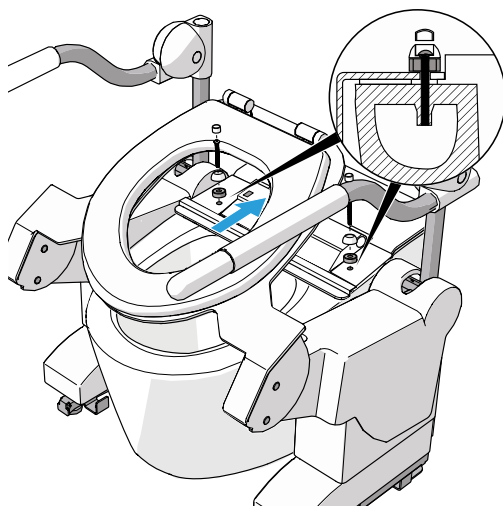

12. Lower the toilet lift to the lowest position by pushing the arm support down-button 19.
13. Check if the toilet seat rests on the toilet bowl.
If necessary, turn the levelling feet to adjust the height.
14. Lower the foot covers and fasten the velcro connections.

5.6. Add-ons


Add-ons such as the hand remote control and/or foot control will, in most cases, be delivered pre-installed on the toilet lift. If this is not the case, refer to the service manual for installation instructions on www.aeroletnl.com.

5.7. Adjusting


5.7.1. Adjusting the rise height and tilt angle

The rise height and tilt angle of the toilet lift seat can be adjusted to suit the end user's height and the toilet's height. Follow these steps for proper adjustment:


1. Connect the power plug 4 to a power outlet.
2. Press the arm support up-button 18 to raise the toilet lift. Stop when the toilet seat reaches an angle of approximately 30 degrees.

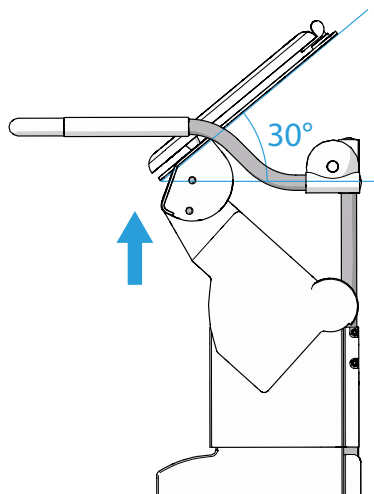


3. Remove the top cover locking screws **3** on both sides.
-

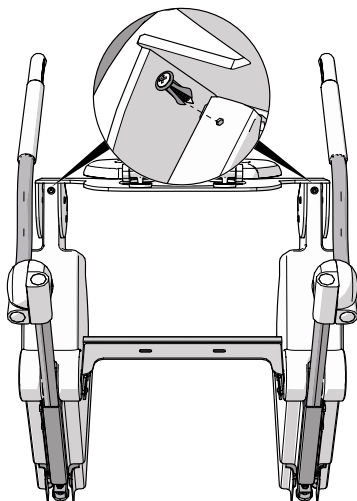

4. Open the velcro connections and remove the top cover **7**.
-

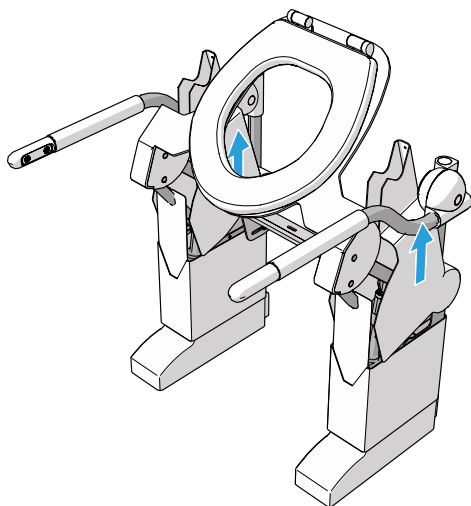

5. Open the velcro connections and remove the middle covers **9** and the foot covers **10** from both sides.
-

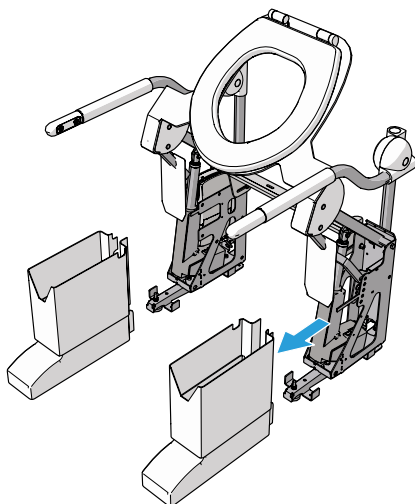


Each foot frame has five hinge positions for adjusting the rise height and tilt angle.

NOTICE



Scan the QR code to choose a suitable hinge position for your preferred rise height and tilt angle.

6. Release the locking springs **25** on both sides.

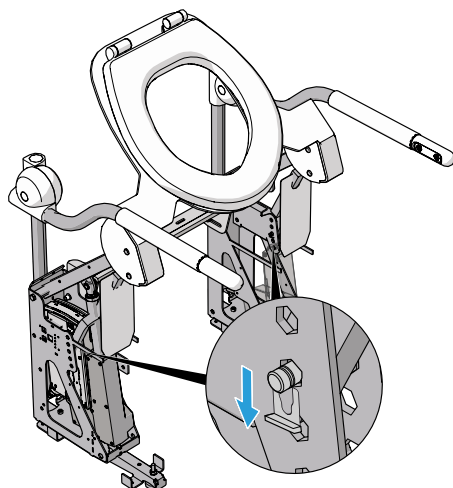


7. Hold the seat frame and pull out the hinge pins **26** on both sides. The seat frame tilts backwards. Place the hinge pins in the positioning holes **24** of your choice. Ensure the hinge pins are in the same positioning holes on both sides.

⚠ WARNING


Hand entrapment hazard. The seat frame will tilt backwards when the hinge pins are removed. Handle carefully.

⚠ CAUTION

Risk of product damage. Use the same hinge hole positions on both sides. Uneven hinge pin positions may result in serious mechanical damage to the toilet lift.

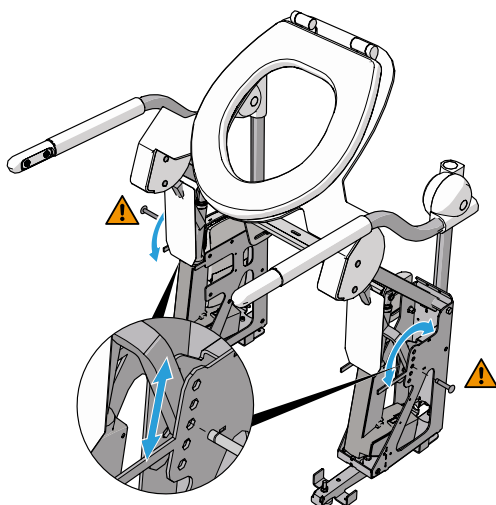

8. Press and hold the toggle button **22**, located on the left drive unit **11**, to update the software with the new tilt angle and height. Press and hold the toggle button until the LED **20** for your chosen position hole number lights up. For example, position hole 3 corresponds to LED 3.
-

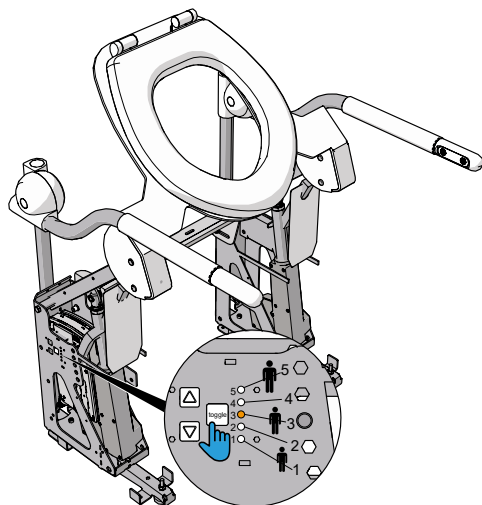

9. Check if the lit LED matches your chosen position hole. If the LED matches, the software is updated to the new tilt angle and height. If the LED does not match the position hole, try again or refer to Chapter 7. Troubleshooting for further instructions.


5.7.2. Arm support height

The fixed armrest and the folding armrest can be adjusted in height.

1. Loosen the two allen bolts **8**.
2. Adjust the armrest to the desired height and tighten the allen bolts.

⚠ CAUTION


Risk of damage to the armrests. Never adjust the armrests to a position where the toilet seat touches the armrests. Always leave enough room for the armrests to move up and down to prevent surface scratches.

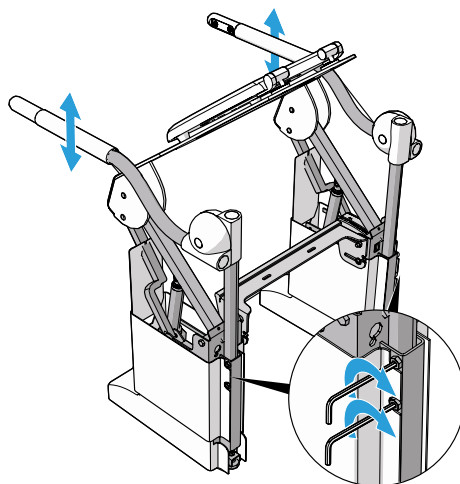

5.7.3. Arm support angle

The angle of the arm support controls can be adjusted.

1. Loosen the allen bolts at the bottom of the arm supports with an allen key.

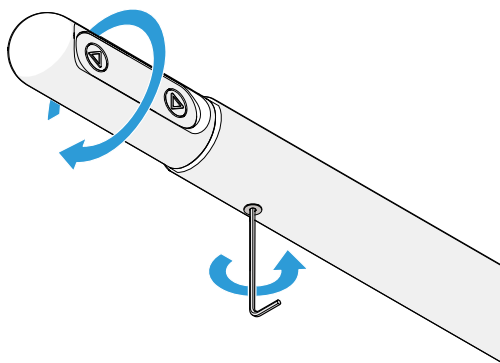

-
2. Rotate the arm support controls in the desired position.
 3. Tighten the allen bolt.

6. USE

Risk of product malfunction. Never exceed the maximum weight capacity. Overloading can cause the lift to fail. Check the manufacturer's specifications.



CAUTION

Risk of product damage. High temperatures can cause overheating. Never operate the lift in temperatures above 40 degrees Celsius.

Risk of product damage. Excessive humidity can cause electrical issues and product malfunction. Never operate the lift when humidity exceeds 60%.



6.1. Lifting and lowering with drive unit controls

The toilet lift can be operated with buttons located on the drive control unit, in case the armrests are out of order during installation, maintenance or service.




1. To lift the toilet lift, press the drive unit up-button , located on the left drive unit.
2. To lower the toilet lift, press the drive unit down-button , located on the left drive unit.

6.2. Resetting

If the toilet lift makes a beep sound when one of the control buttons is pressed, reset the lift. Follow the following procedure to reset the toilet lift:

1. Press the arm support up-button  and down button  simultaneously and hold until the beep sound dies down.
2. Lower the toilet lift by pressing the arm support down-button. Bring the toilet lift at the lowest position.
3. Contact your dealer.

7. TROUBLESHOOTING

Problem	Possible cause	Solution
The toilet lift does not lift or lower.	The toilet lift is not connected to a power supply.	Connect the toilet lift to a power supply.
	Left, right control unit or both control units are defective.	Contact dealer.
	Arm support up-button and arm support down-button are pressed simultaneously.	Press arm support up-button or arm support down-button. Do not press simultaneously.
	The toilet lift is out of order for installation, maintenance or repair.	Use the drive unit up and down button   to lift and lower the toilet lift.
The toilet lift does not lift or lower and all 5 LEDs  on the left drive unit are flashing.	Communication error in one of the arm support control buttons.	Reset the lifter and lower the toilet lift. Contact your dealer.
The toilet lift makes a beep sound when one of the control buttons is pressed.	The toilet lift has an electronics error.	Reset the toilet lift. See chapter 6.2..
The toilet lift makes a squeaking noise when the toilet lift is lifted or lowered.	The guide rods are greasy and/or dirty.	Clean and degrease the guide rods with a degreaser such as benzene.

8. MAINTENANCE

8.1. Repair

⚠ WARNING

Improper maintenance hazard. Risk of serious injury and product malfunction. Always repair the device by trained and authorized personnel. If the toilet lift is not repaired by an authorised body, there is a risk of malfunction and the warranty may be withdrawn.

⚠ WARNING

Risk of electrical shock. Disconnect the power supply before performing any maintenance. Always unplug the device first.

Improper maintenance hazard. Risk of serious injury and product malfunction. Always repair the device by trained and authorized personnel. If the toilet lift is not repaired by an authorised body, there is a risk of malfunction and the warranty may be withdrawn.

Repairs should be done by trained and authorized personnel. If repairs are done under warranty, send defective parts to Aerolet.


For repairs to the electronics, consult the electrical diagram in Attachment 2.

It is possible to order spare parts on request. Contact Aerolet for a parts list.

9. DECOMMISSIONING

In case of permanent or prolonged non-use, or for transport of the product, the following procedure should be followed to decommission the toilet lift.

1. Press the arm support up-button **18** to bring the toilet upwards.
2. Unscrew the carriage bolts from the connecting bracket **6**.

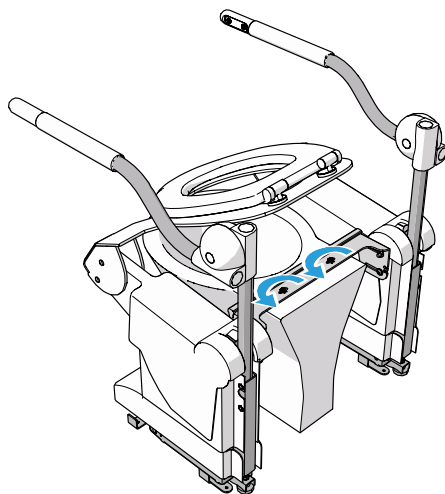

3. Disconnect the power plug **4** from the power outlet.
4. Lift the toilet lift by the arm supports **1** **5** and remove the toilet lift.

10. WARRANTY

Aerolet guarantees the Aerolet® vertical toilet lift against material and manufacturing defects for a period of two years from purchase by the end user. The warranty period starts either on the date of delivery, or on the installation date, when the installation has been carried out by Aerolet or by an authorised dealer. During this warranty period, Aerolet or an authorised dealer will, at its discretion, repair or replace any defective parts. Parts repaired under warranty should be returned to Aerolet.

The warranty on your Aerolet toilet lift does not apply to defects resulting from the following:

- Improper or inadequate maintenance by the user.
- Unauthorised adjustments and/or modifications or misuse.
- Use outside the environmental specifications for which the product is intended.

If the toilet lift is not opened by an authorised body, there is a risk of malfunction and the warranty may be withdrawn.

11. DISPOSAL



Separate and dispose the components of the machine into the applicable waste streams based on their materials, in accordance with local regulations.

ATTACHMENTS


Attachment 1. Declaration of conformity

Aerolet declares that the Aerolet® vertical toilet lift complies with the following directive: Medical Devices Regulation (EU) MDR 2017/745.

The product conforms to the following standards:

- EN 60601-1:2006
- EN 60601-1-2:2015
- EN-ISO 14971:2019+ A11:2021
- EN-ISO 13485:2016
- EN 1041:2008+A1:2013
- EN 6060-2
- NEN-EN 21856

Attachment 2. Electrical diagram

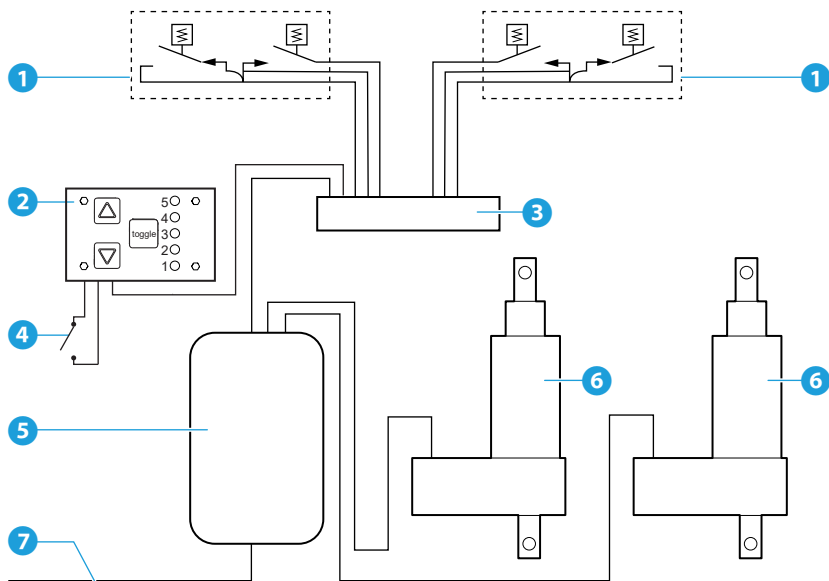

- ① Arm support control
- ② Toggle circuit board
- ③ Splitterbox
- ④ Limit switch

- ⑤ Control box
- ⑥ Actuator
- ⑦ Power cord



Aerolet

Het Lentfert 38, 7547SR Enschede

info@aerolet.nl

www.aerolet.nl - www.aeroletnl.com

Original user manual

v1.0 - 02/2025